EN ⚠ Warnings:

This product is recommended for personal use only, especially the cannulas. Use the device and therapeutic solution at body temperature as advised by your physician. Use in children and pregnant women only if prescribed by a physician. Assistance must be provided to persons with reduced mental and/or physical capabilities when using the device. Keep the product out of reach of children, it is not a toy. Choking hazard. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the Competent Authority of the Member State in which the user and/or patient is established. Dispose of the product in accordance with applicable local regulations.

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EN Valve
SLO Ventil
HR Ventil

EN Mediblink Enema Irrigator M099

Intended Use: The irrigator is a medical device designed for intestinal rinsing by enema and for vaginal irrigation.

Content: 2-liter pouch with hanger, 1.4 m flexible tube with valve, short hard cannula for anal irrigation, long hard cannula for vaginal irrigation, storage bag. The product does not contain latex.

Instructions for use: Before first use, wash all parts with soap and rinse with water. Immerse the anal or vaginal cannula in boiling water for 5 minutes. **1** Attach the tube with valve to the 2 L pouch by rotating it until secure. **2** Insert the valve into the tube. Select the appropriate cannula depending on the intended use, and screw it firmly onto the valve to prevent leakage. **3** Ensure the valve is closed (handle positioned perpendicular to the outlet). Fill the pouch with tepid liquid. The liquid must completely fill the tube to avoid air bubbles during irrigation. To achieve this, slightly open the valve to let some liquid flow out, then close it again. **NOTE:** The flow pressure depends on the height difference between the cannula and the pouch. The pouch can be hung at the desired height but never below the cannula, as this may cause backflow. **4** Before each reuse, repeat the procedure described in steps 1–3.

After use: **1 Cleaning:** After each use, clean the pouch, the used cannula, the tube, and the valve with soap and water and rinse them thoroughly. We strongly recommend disassembling and thoroughly cleaning all parts, including the inside of the cannulas and valve. **2 Storage:** Allow all parts to dry completely before storage. Store the device in a clean, dry, and hygienic place.

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