Mediblink®
medical devices

Electric Nasal Aspirator SuperEasy M405

Mediblink®

3 YEARS
WARRANTY

www.mediblink.com

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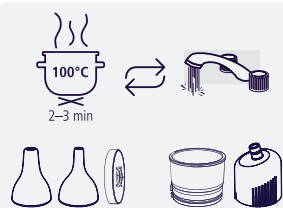
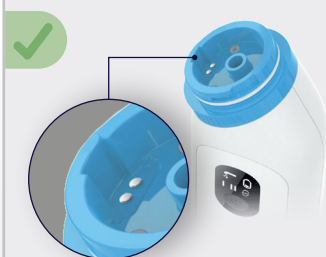
EN Unique patented design safely holds
fluids in a container without spilling.

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
EN Longer life span with automatic shut-
down in case of over-flow.

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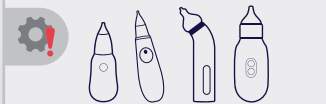HR Nosni aspiratori bez senzora preliivanja
brzo se znaju pokvariti.

EN Mediblink Electric Nasal Aspirator SuperEasy M405

Intended Use: The device which is a portable, handheld, reusable, battery-powered suction device, is intended to enable an adult to gently suction and clear excessive mucus from the nasal passages of an infant or child to facilitate easier breathing in home care. The aspirator features a handy design and is both safe and easy to use. Two different sizes of soft silicone suction nozzles enable safe product use. Aspirator offers highly efficient suction with three speed levels and pulse suction technology, automatic shut-down in case of overflow to ensure long-lasting device use, a patented anti-backflow design for clean and hygienic use and a child-friendly, low-noise suction function (<60dB(A)). The suction nozzles, collection cup and cover are removable and can simply be cleaned under running water and to disinfect. The warranty period is three years. **Content:** 1 Nasal aspirator, 2 Suction nozzles (1 wide, 1 narrow), 2 Sealing rings, 1 User manual, 3 AAA alkaline batteries.

Warnings: The nasal aspirator is intended only for the suction of nasal secretions. Please do not use it for other purposes. If the user's nose is dry, it should be humidified first. Keep the suction nozzles clean and sanitary when in use. Keep the aspirator out of the reach of children under 12 years old. Potential choking hazard. Do not immerse the main part of aspirator in water or other liquids, and do not rinse it directly under the tap. Read the instructions for use before using the aspirator.

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Mediblink® is a registered trademark.
©Packaging design is copyright.
No infringements of our trademarks
and copyright material is allowed.
www.mediblink.com

Hetaida Technology Co., Ltd.
Room 801, 802, 803, 804, 901, 2# Building Scientific Research
Center No.6 Minfu Road, Liaobu Town, Dongguan City,
Guangdong Province, P.R.China

e-FU: <http://www.mediblink.com/f/m405.pdf>

1.0, 30.5.2025

LOT

REF
M405