1

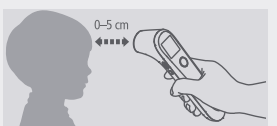
- EN** Press the power/test button and wait until display is activated
- SLO** Pritisnite gumb za vklop/merjenje in počakajte, da se zaslon aktivira
- HR** Pritisnite gumb za napajanje/testiranje i pričekajte da se zaslon aktivira

2


- EN** Select the appropriate age group
- SLO** Izberite ustrezno starostno skupino
- HR** Odaberite odgovarajuću dobnu skupinu

3

- EN** Aim at the forehead/object with a distance less than 5 cm and press test button
- SLO** Usmerite v čelo/predmet z razdalje manj kot 5 cm in pritisnite gumb za merjenje
- HR** Usmerite u čelo/predmet s udaljenosti manje od 5 cm i pritisnite gumb za testiranje

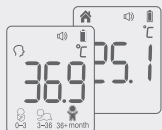4

- EN** Read the body/object temperature
- SLO** Odčitajte temperaturo telesa/predmeta
- HR** Očitajte temperaturu tijela/predmeta

e-FU: <http://www.mediblink.com/f/m330.pdf>

UDI

Mediblink
medical devices

Infrared Forehead Thermometer Age Accuracy™

Fast & Accurate

Body Temperature

Object, Liquid & Ambient Temperature

Sound ON/OFF

NEW

Age Accuracy™

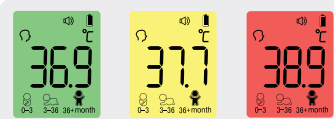
Select patient's age for accurate measurement

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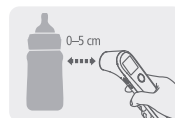
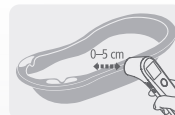
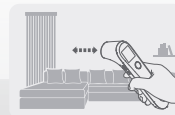
EN Mediblink Infrared Forehead Thermometer Age Accuracy™ M330

Intended Use: The infrared forehead thermometer Age Accuracy™ is intended for the intermittent measurement of human body temperature from the skin surface of forehead. The device can be reused by people of all ages for home and clinical use. It is fast, has an age setting option for accurate measurement, silent/night mode and can store the last 30 measurements. Reference body site: oral. This thermometer converts the forehead temperature (range: 34.0–43.0°C) to display its "oral equivalent". Operating mode: Adjust mode (Forehead mode). It can also measure temperature of object, liquid & ambient (range: 0–100°C). Warranty is 3 years. **Content:** 1x Infrared forehead thermometer, 1x Storage case, 1x Instructions for use, 2x 1.5V AAA batteries.

Warning: Do not allow children to take their temperature unsupervised, some parts are small enough to be swallowed. Do not touch the temperature probe with hands. Never immerse this device in water or other liquids.

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JOYTECH Healthcare Co., Ltd.
No.365, Wuzhou Road,
311100 Hangzhou, Zhejiang Province,
PEOPLE'S REPUBLIC OF CHINA
E-mail: info@sejoy.com
Shanghai International Holding Corp. GmbH (Europe)
Eiffelstrasse 80, 20537 Hamburg, Germany
E-mail: shholding@hotmail.com

CE 0123

1.0, 1.3.2025

6 942284 605820