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
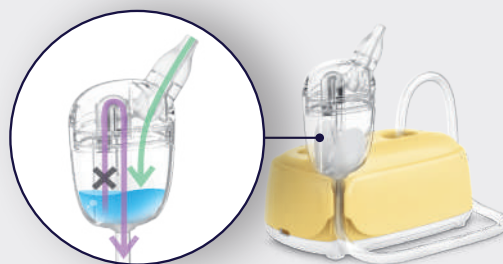
EN SuperEasy PRO is designed to ensure that liquid (mucus) cannot pass beyond the collection cup into the connecting tube or the device's motor. This extends the device's lifespan, ensures reliable operation, and provides an excellent user experience.

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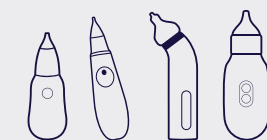EN Nasal aspirators without overflow protection technology break down quickly. Liquid (mucus) can easily enter the motor, causing damage or even destroying the aspirator.

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EN Mediblink Electric nasal aspirator SuperEasy PRO M410

Intended Use: The device which is a portable, handheld, reusable, battery-powered suction device, is intended to enable an adult to gently suction and clear excessive mucus from the nasal passages of an infant or child to facilitate easier breathing in home care. The aspirator features a handy design and is both safe and easy to use. Two different sizes of soft silicone nozzle tips enable safe product use. Aspirator offers highly efficient suction with two speed levels, a child-friendly, low-noise suction function (<50 dB) and music function. Compared to regular battery-powered aspirators, the SuperEasy PRO is designed to effectively prevent nasal mucus from entering the motor. This extends the lifespan of the device, ensures reliable performance, and provides an excellent user experience. The suction nozzle tips, collection cover and cup are removable and can simply be cleaned under running water and disinfected. The warranty period is three years. **Content:** 1 Nasal aspirator, 1 Collection cup, 1 Connecting tube (80cm), 2 Suction nozzle tips, 2 Sealing rings, 1 Storage case, 1 User manual, 2 AA alkaline batteries and 1 USB-C cable.

Warnings: Nasal aspirator is only suitable for sucking nasal secretions. Please do not use it for other purposes. If the user's nose is dry, it should be humidified first. Keep the nozzle tips clean and sanitary when in use. Keep the aspirator out of the reach of children under 8 years old. Potential choking hazard. Do not immerse the main part of aspirator in water or other liquids, and do not rinse it directly under the tap. Read the instructions for use before using the aspirator.

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Vapo Healthcare Co., Ltd
Southern Unit of the Third Floor Building B,
No. 99 Yu Dai West Rd, High Tech District,
Kunshan, Suzhou, Jiangsu Province, P.R.China

EU REP

Am Schulzentrum 12, 41564 Kaarst, Germany

e-FU: <http://www.mediblink.com/v/m410.pdf>

Mediblink® is a registered trademark.
©Packaging design is copyright. No infringements of
our trademarks and copyright material is allowed.
www.mediblink.com

Manufactured for (importer for
EU & distributor) / Hergestellt für
(importeur für EU & distributor) /
Proizvedeno za (uvoznik za EU &
distributor): Mediblink d.o.o.,
Gubčeva cesta 19, 8210 Trebnje,
Slovenija

1.0, 30.9.2025