EN Oraflago[®] pro ORIGINAL GEL
Oraflago[®] pro ORIGINAL GEL is a fast and effective solution for the protection and regeneration of oral mucosa. Once applied, it creates a protective layer that soothes irritated oral tissue and provides relief from discomfort. It is intended for anyone who wants comprehensive care and protection for sensitive areas of the oral cavity, even when using dental appliances.

PRODUCT DESCRIPTION

Oraflago[®] pro ORIGINAL GEL is a fast and effective solution for the protection and regeneration of the oral mucosa. Each application creates a protective layer that soothes irritated areas, relieves discomfort and protects sensitive areas inside the oral cavity. The thicker texture allows the gel to remain in place longer, ensuring long-lasting protection and increased effectiveness. It contains ten carefully selected active ingredients and hyaluronic acid to intensively moisturise tissues, promote natural renewal and accelerate the regeneration of the oral mucosa.

INDICATIONS

Oraflago[®] pro ORIGINAL GEL is suitable for local application to various oral tissue changes and sensitivities, such as sensitive or irritated gums, minor mouth injuries, irritation after surgical procedures, and discomfort from dental appliances or poorly fitting dentures.

INGREDIENTS

Purified water, Macadamia seed oil (Macadamia ternifolia), Horse chestnut extract (Aesculus hippocastanum), Tocopheryl acetate (Vitamin E), Asian pennywort extract (Centella asiatica), Triethanolamine, Carbomer, Fenugreek extract (Trigonella foenum-graecum), Aloe vera gel (Aloe barbadensis), Imidazolidinyl urea, Sodium hyaluronate, Phenoxethanol, Sodium dehydroacetate, Alanitol, Glycerol essential oil (Cyperus sempervirens), Tea tree essential oil (Melaleuca alternifolia), Thymine essential oil (Thymus zygis), Ubiquinone (Coenzyme Q10), Propylene glycol, Ascorbic acid (Vitamin C), Panthenol (Provitamin B5), Ethylparaben, Methylparaben, Propylparaben.

INSTRUCTIONS FOR USE

Wash your hands and dry the sensitive area. Apply a small amount of gel to your finger or a cotton swab and gently spread it over the sensitive area. Do not eat or drink for at least 30 minutes after application. Apply 3–5 times daily, preferably after brushing your teeth. The gel is safe for regular use. Oraflago[®] pro ORIGINAL GEL is hygroscopic, which means it absorbs water. To ensure optimal performance, we recommend drying the affected area with a cotton swab or paper towel before application to allow Oraflago[®] pro ORIGINAL GEL to work faster. Reapply the gel after drinking or eating.

PRECAUTIONS:

Oraflago[®] pro ORIGINAL GEL is intended for local use on the oral mucosa. Keep out the reach of children. Make sure that the child does not swallow the cap. Do not apply the gel to the back of the tongue or throat.

Should you have any hypersensitivity or allergic reactions to any of the ingredients, we advise you to consult your doctor or pharmacist. Avoid contact with eyes. In the event of accidental contact, rinse thoroughly with water.

STORAGE CONDITIONS

Store in a dry, dark place at a temperature below 25 °C. We recommend using Oraflago[®] pro ORIGINAL GEL within 3 months after the package is first opened.

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