TONIMER

PANTHEXYL BABY SPRAY 800 m0sm/kg

EN_INDICATIONS

Sterile hypertonic seawater solution to remove and fluidify the mucus in the nasal passages. Indicated for newborns and children suffering from colds with microbial superinfection (such as rhinopharyngitis and rhinosinusitis) and the prevention of acute otitis media caused by the stagnation of mucus. Enriched with xylitol and panthenol, it hygienizes, protects, soothes, moisturises and decongests the mucous membranes, thus facilitating more comfortable breathing. It contains no preservatives and does not dispense propellant (air), as it is not in contact with the product, thus reducing the possibility of allergic reactions.

It is supplied with a non-return valve and a sac that isolates the solution, which:

- guarantees sterility;
- makes it possible to use the bottle in any direction (360°);
- prevents mucus from being sucked into the bottle.

DIRECTIONS FOR USE:

Wash your hands before use.

1. Once the cap is removed, children who are able to use the product on their own (under adult supervision) can do so by tilting the cylinder at any angle. It is best applied by leaning over a washbasin, tilting the head to one side and inserting the spray nozzle into the uppermost nostril. Press on the nozzle for about 5 seconds to release the product. Repeat the same procedure in the other nostril (drawing 1).
2. For newborns and children requiring adult assistance: lie the child on its side, insert the spray nozzle into the uppermost nostril and press for about 5 seconds. Turn the child onto its other side and repeat the same procedure in the other nostril (drawing 1).
3. After use, always wash and dry the nozzle and close the container using the cap supplied.

FREQUENCY OF USE:

- 0 to 3 months: 1-2 times a day for 15 days, or according to medical advice;
- after 3 months: 3-4 times a day for 15 days, or according to medical advice.

WARNINGS: for nasal use. For personal use only. Do not use the product if the bottle appears to be damaged. Do not use the product after the expiry date. The expiry date refers to an intact, correctly stored product. Do not exceed the recommended dosage. Do not use the product if you know you are sensitive to any of its components. In rare cases, temporary

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burning of the mucous membranes or slight nasal bleeding can occur. This will not compromise the safety and efficacy of the product. Stop using the product, consult your doctor and inform the manufacturer if side effects occur. If symptoms persist after treatment, consult your doctor. The product should only be used on newborns under paediatric advice. Drugs and medical devices for inhalation must only be taken simultaneously with the product under the supervision of a doctor who can assess possible interactions. In the event of serious accidents related to administration of the product TONIMER PANTHEXYL BABY SPRAY, please send a report to the manufacturer (customercare@ganassini.it), the Italian Ministry of Health or the competent authority of the EU member state or of the country in which the product was used. Avoid contact with the eyes; in the event of contact, rinse with water. Keep the product out of the reach and sight of children. Dispose of the device responsibly after use. **Store with the cap closed, away from heat, in a cool and dry place. Warning** Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children. Do not make use other than those specified.

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ISTITUTO GANASSINI S.p.A. di Ricerche Biochimiche